FRIWO

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Netzteile

Instruction Manual

Power Supplies

DT 150M

FW 7333SM, 7362M, 7401M, 7405M, 7488M, 7555M,
7556M, 7660M, 7662M, 7721M, 8000M, 8001M,
8002M, 8002.1M, 8004M, 8030M, 8030M/DT,
8060.1M

NEO 006.0-I, 012.0-I, 018.0-I, 030.0-I, 060.0-I, 060.0-D

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FOR YOUR SAFETY

Read instruction manual

This manual contains information for the safe handling of those power supplies which are listed on the front page.

It explains the installation of the mains and secondary adapters of the power supply.

- Always read the instruction manual first before using the power supply.
- The instruction manual is part of the product and should always be at hand.
- Illustrations show functions by way of example and may differ from the actual look of the power supply.

Prevent shock-hazard

- Never use a power supply with damaged housing or cable.
- Never modify a power supply or open its housing.
- Connect the power supply to a regular outlet only.
- Protect connectors from moisture.

Prevent property damage

- Never carry the power supply by its electrical cable or pull at the cable.
- For separating the connection always pull on the plug, never on the cable.
- Cables must be installed safely to avoid tripping over.
- Disconnect the power supply during thunderstorms.
- Only use original FRIWO spare parts or accessories.

Observe all symbols

Symbols for your safety and for environmental protection

	Read instruction manual
	Unit DT150M with protection class I Electrically conductive housing parts are connected to a protective conductor system.
	Units with protection class II AC input and DC output are electrically safely isolated
	Indoor use only.
	Dispose the power supply via the designated take-back system
	To identify a type CF applied part complying with IEC 60601-1. (see page 28 / FW7660M/FR/09)

DESIGNATED USE

The power supply feeds medical products in unsterile environments, in professional healthcare facility and home healthcare environment.

- For the connection of power grids (**AC** input) and the **DC** output of devices which are compatible with the technical characteristics of the power supply.
- Use only at permissible ambient conditions, temperature, Atmospheric pressure and humidity. → TECHNICAL DATA
- Use only in expanded patient environment.
- Only FW7721M: Suitable for use in direct patient environment.
- Only DT150M: Suitable for use in professional healthcare environment.

Restrictions for all power supplies:

- Never use for life-saving or life-sustaining equipment.
- Never use for equipment whose malfunction or failure may cause injuries or lead to death.
- Never use in an oxygen-enriched environment.

- Do not use with flammable anesthetics and not in conjunction with highly flammable substances.
- For indoor use only.
- Do not use in rooms or areas with high intensity of electromagnetic disturbances (e.g. RF shielded room of an ME SYSTEM for magnetic resonance imaging or near active HF SURGICAL EQUIPMENT)
- Prohibited environmental conditions:
 - Constant vibrations.
 - Radical changes in temperature.

Danger of fatal electric shock!

Use power supply in dry environment only. Avoid splashing water. Do not immerse in liquids.

CONNECTING OF ADAPTER

AC input (mains adapter):

DT 150M	Power supply with C14 adapter. → MOUNTING
FW 7401M, 7405M, 7488M, 8004M, 8030M/DT NEO 060.0-D	Power supply with C8 adapter. → MOUNTING
FW 7362M, 7555M, 7556M, 7660M, 7662M, 7721M, 8000M, 8001M, 8002M, 8002.1M, 8030M, 8060.1M NEO 006.0-I, 012.0-I, 018.0-I, 030.0-I, 060.0-I,	Power supply with inter- changeable mains adapter for different regions. → MOUNTING

NOTE

FW 8000M, 8001M, 8002M, 8002.1M, 8030M, 8060.1M

NEO 006.0-I, 012.0-I, 018.0-I, 030.0-I, 060.0-I

Protection against Vertically dripping water (IP42)

- only with mounted special IP42 interchangeable mains adapter → MOUNTING
- Consider Position of use → POSITION OF USE

DC Output:

Different secondary adapters (coax plugs or jacks) can be connected to the DC output. The polarity is set by rotating the plug. → MOUNTING

NOTE

Reversed polarity (plus and minus) can damage the electronics of the connected device!

- Observe polarity signs on the plug prior to connecting.
→ MOUNTING

NOTE

The combination of secondary adapter and power supply can not be disconnected!

- Connect only when plug is to be permanently mounted.

WARNING

Risk of Strangulation when a baby or child is involved in the power supply cable

- Keep the input and output cables out of reach of babies or children.

WARNING

Use of cables or adapters other than those specified or provided by Friwo could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation

OPERATION

Display of operation

- If the LED lights up, the power supply is ready for use.

WARNING

Electric equipment can significantly impact the performance of the power supply!

- Do not place other electric equipment close to the power supply
- Keep power supply and medical device at least 30 cm apart from HF communication equipment (for example antenna cables, external antennas).

CAUTION

Potential risk of burns during operation or in direct exposure to skin!

- Do not touch power supply during operation.
- Avoid direct contact with patient.
- Exception: FW7721M is suitable for direct contact with patient.

MAINTENANCE

Do not use chemical cleaning agents.

WARNING

Risk of lethal electric shock if fluid leaks into the power supply!

- Disconnect power supply before cleaning.
- Do not use fluids for cleaning.
- Wipe with a dry cloth only.

FAULT CORRECTION

The power supply is maintenance-free. In case of a faulty power supply or malfunctions that can not be solved, please contact the dealer or FRIWO.

Disposal

NOTE

Environmental contamination caused by incorrect disposal!

- Cut off the cable of the power supply and dispose the unit via the designated take-back system. → MOUNTING

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